

Operating instructions

1、 Start Interface

The interface shows the brand and the battery current capacity.




Display the brand.

Display the current capacity of battery inside BPM.
When the battery is lower than 20%, please
immediately charge the battery.

2、 Main Menu

The main menu of the application. You can switch between measurement and add data by slide left and right.

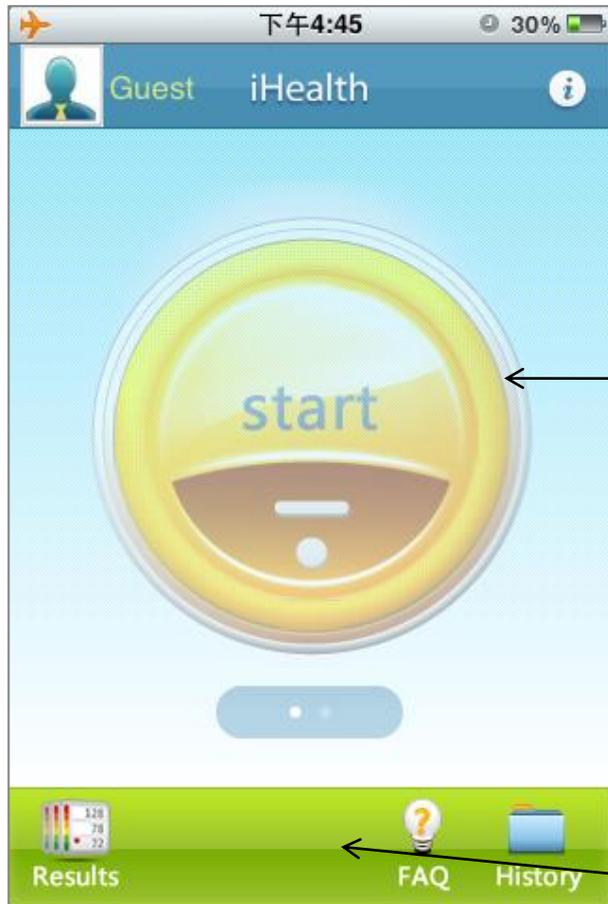


Touch for All Users page.
Touch for Information page.

Touch for Measurement page to start measurement.

- History, touch for History page, management the stored data.
- FAQ, frequency question and answer.
- Results, check the Test Result page.

2、 Main Menu

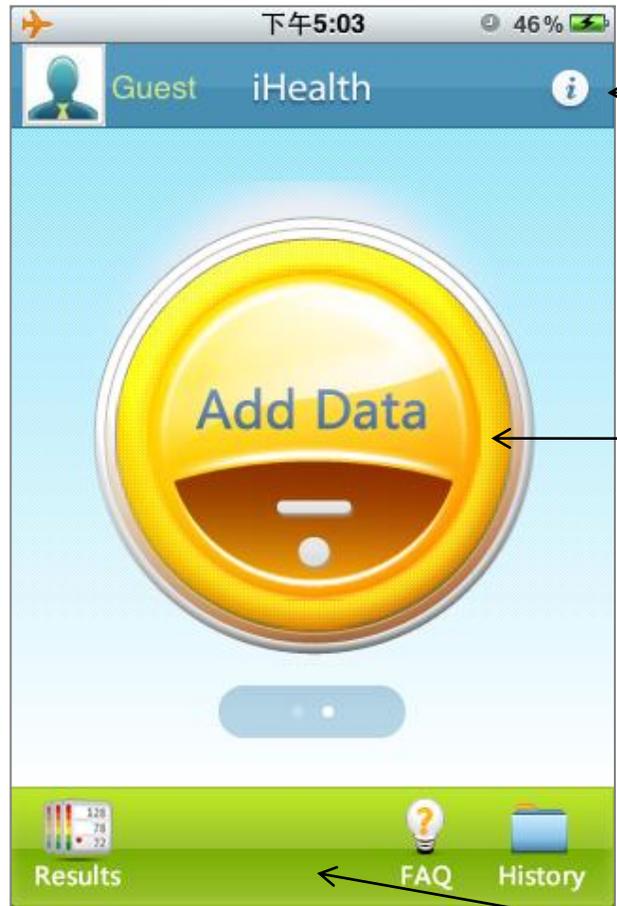


Touch for All Users page
Touch for Information page.

Touch for add data page to add the blood pressure data..

- History, touch for History page, management the stored data.
- FAQ, frequency question and answer.
- Results, check the Test Result page.

3、 All Users

Use to manage the information of users.

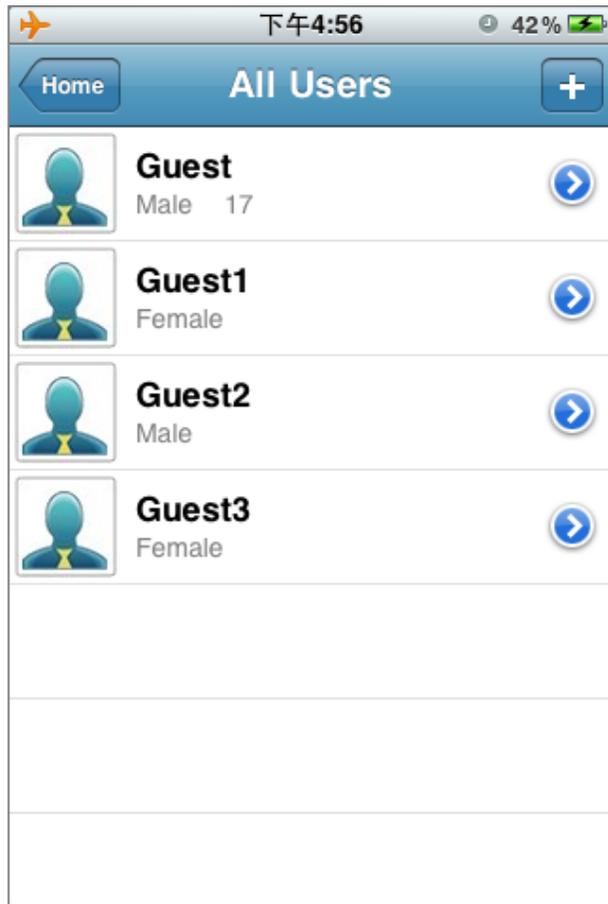


← Home: go to Main Menu page.
“+”: Add new user.

← Display the photo, name, gender and age.
Touch the row for choose the user and back to the main menu..
Touch the accessory button to go to the Detail Info of User page.

4、 Add Users

Use to add new user.

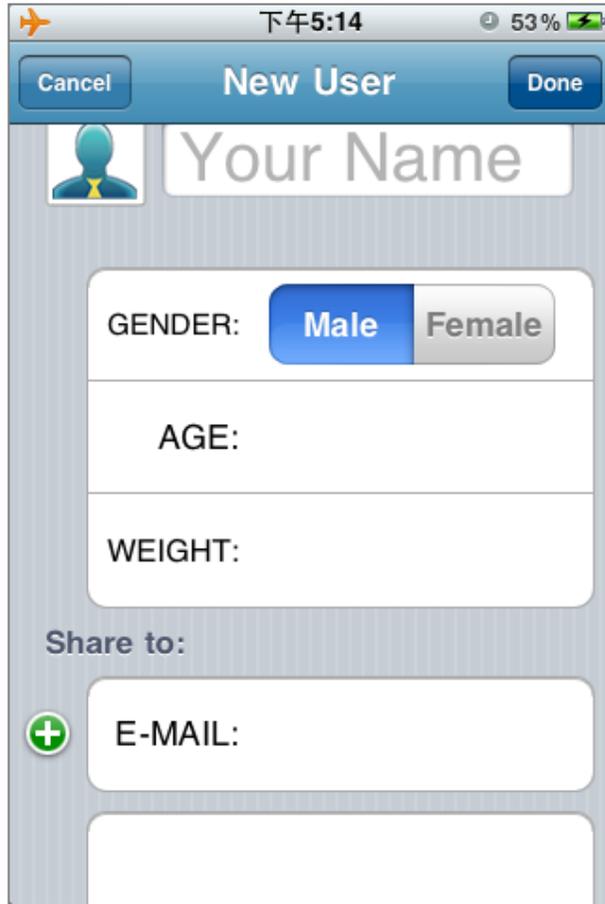

The screenshot shows an iOS-style 'New User' form. At the top, there is a blue header bar with 'Cancel' on the left and 'Done' on the right. Below the header, there is a profile picture placeholder (a blue person icon) and a text field containing 'Your Name'. The form then has three input fields: 'GENDER:' with two buttons labeled 'Male' and 'Female', 'AGE:', and 'WEIGHT:'. Below these is a 'Share to:' section with a plus icon and an 'E-MAIL:' field.


Cancel: Cancel for user added.

Done: Confirm the user settings, and go to Detail Info of User page.

Input the information of the user, such as the photo, gender, age, weight, the share to E-mail and others.

5、 Detail info of User

Use to edit ,choose or delete the user

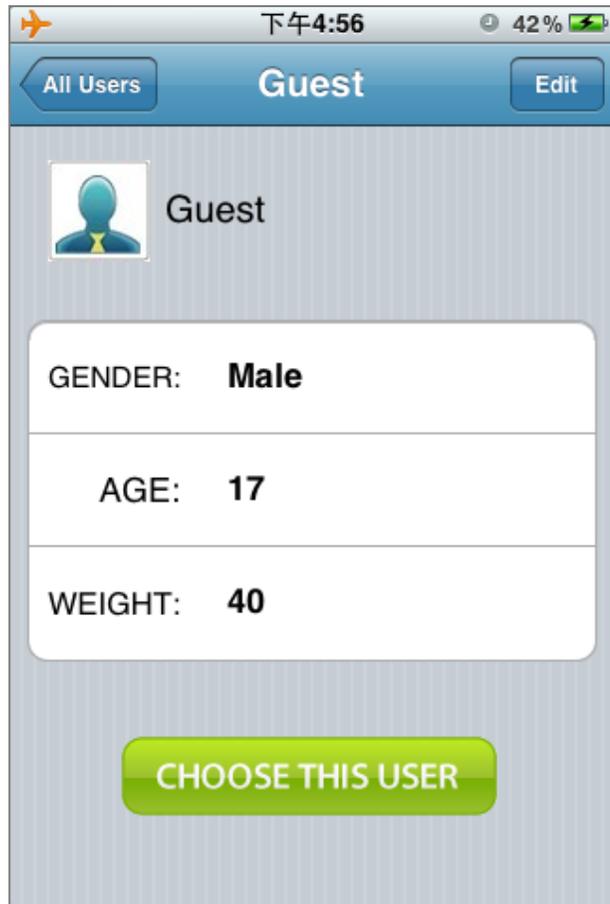

← All Users: Back to the All Users page.
Edit: Edit the detail info of the user.

← Display the information of the user,
such as the photo, gender ,age,
weight, the share to E-mail and others.

← Choose the user and back to the All Users page.

5、Detail info of User

The screenshot shows a mobile application interface for editing user details. At the top, there is a status bar with the time '下午4:56' and battery level '42%'. Below the status bar is a blue header with the name 'Guest' and a 'Done' button. The main content area is divided into sections: 'GENDER:' with 'Male' selected and 'Female' as an alternative; 'AGE: 17'; and 'WEIGHT: 40'. Below these is a 'Share to:' section with a green plus icon and an 'E-MAIL:' label next to a text input field. At the bottom, there is a red button labeled 'DELETE THIS USER'.


← Done: Confirm the user setting.

← Edit the information of the user, such as the photo, gender ,age, weight, the share to E-mail and others.

← Delete the user and back to the All Users page.

6、 Add data

Add data by manual.

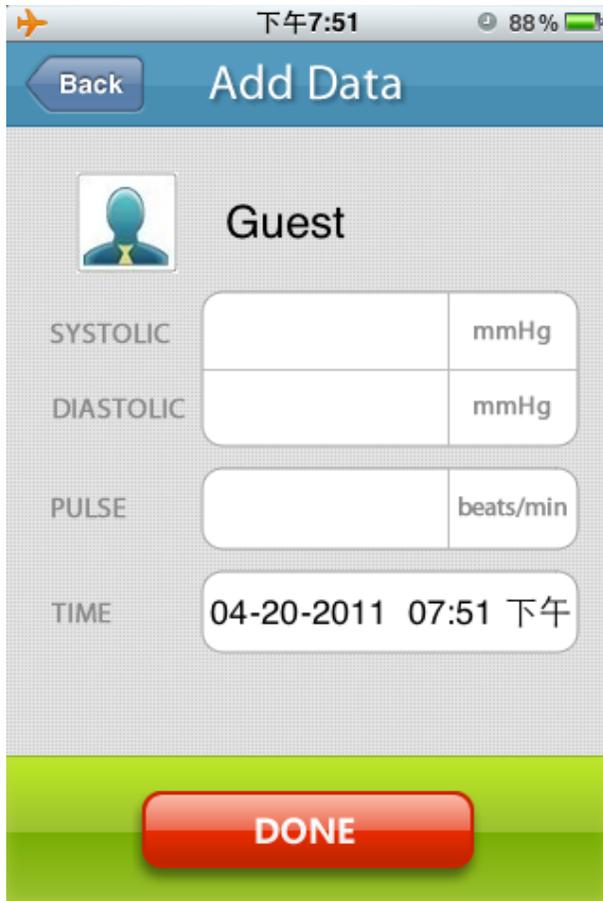

The screenshot shows an iPhone interface for adding data. At the top, there is a status bar with the time '下午7:51' and battery level '88%'. Below that is a blue header bar with a 'Back' button and the title 'Add Data'. The main content area has a grey background with a user profile section for 'Guest' featuring a blue person icon. Below the profile are four input fields: 'SYSTOLIC' (mmHg), 'DIASTOLIC' (mmHg), 'PULSE' (beats/min), and 'TIME' (04-20-2011 07:51 下午). At the bottom, there is a green bar with a red 'DONE' button.


← Back: go back to the last page.
Photo: go to the All User page.

← Input the measurement data and time.

← Save the add data and go to the Test Result page.


7、 Measurement

Use for control the blood pressure measuring process and display the process.

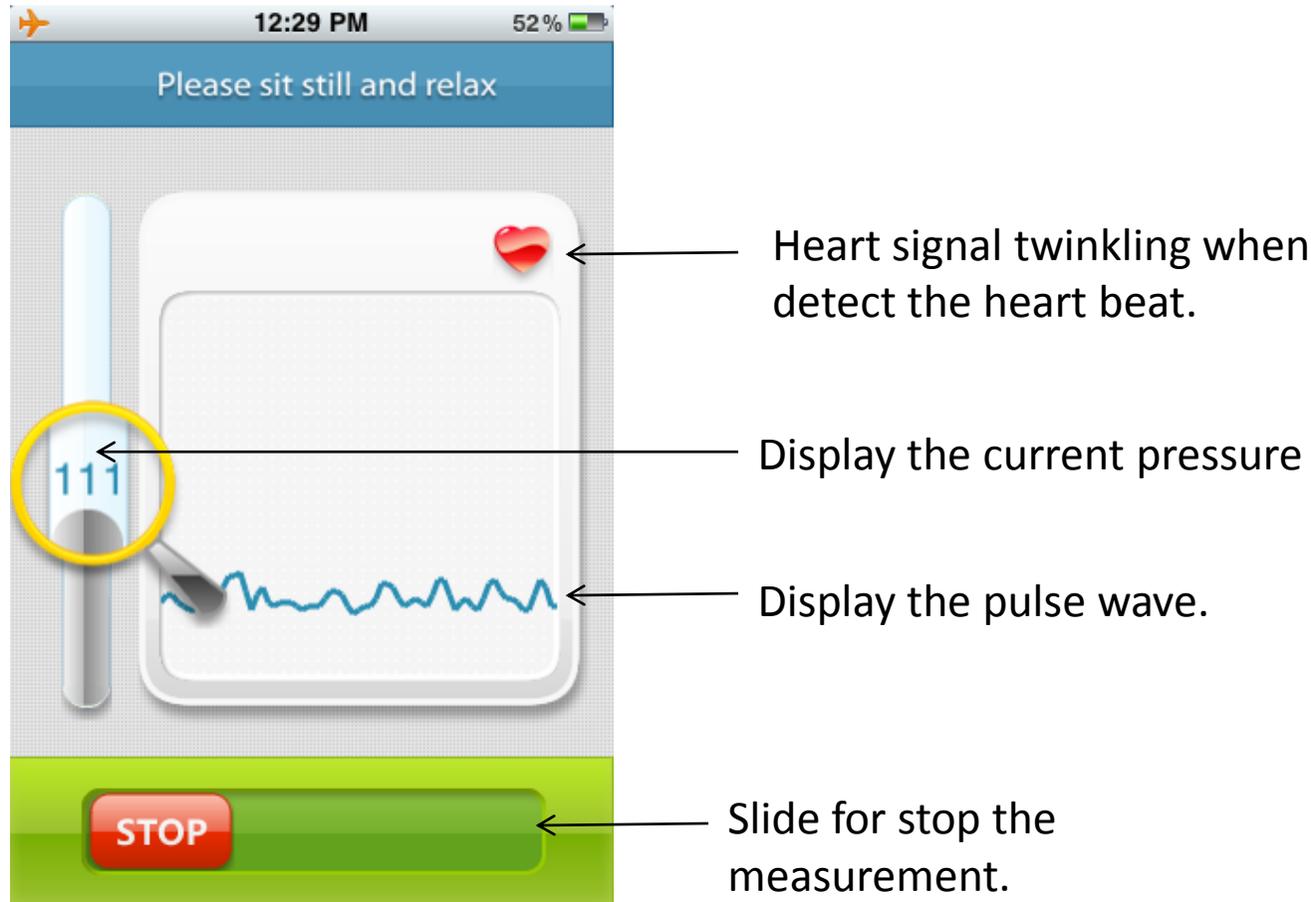

Note: Please consult a health care professional for interpretation of blood pressure measurements.

8、 Test Result

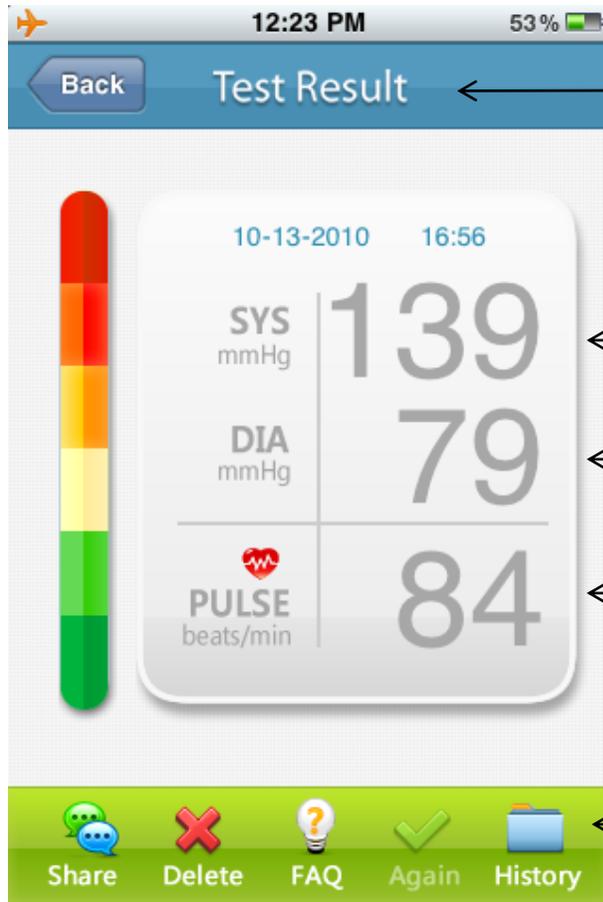

Back: go back to the last page.

Display the Systolic


Display the Diastolic

Display the Pulse rate

- Share: mail the data to the mail address you set.
- Delete: delete the current memory.
- FAQ: go to FAQ page
- Again: measurement again.
- History: go to History Page.




8、 Test Result

Click systolic, diastolic, pulse or WHO classification column respectively, will show the corresponding description.

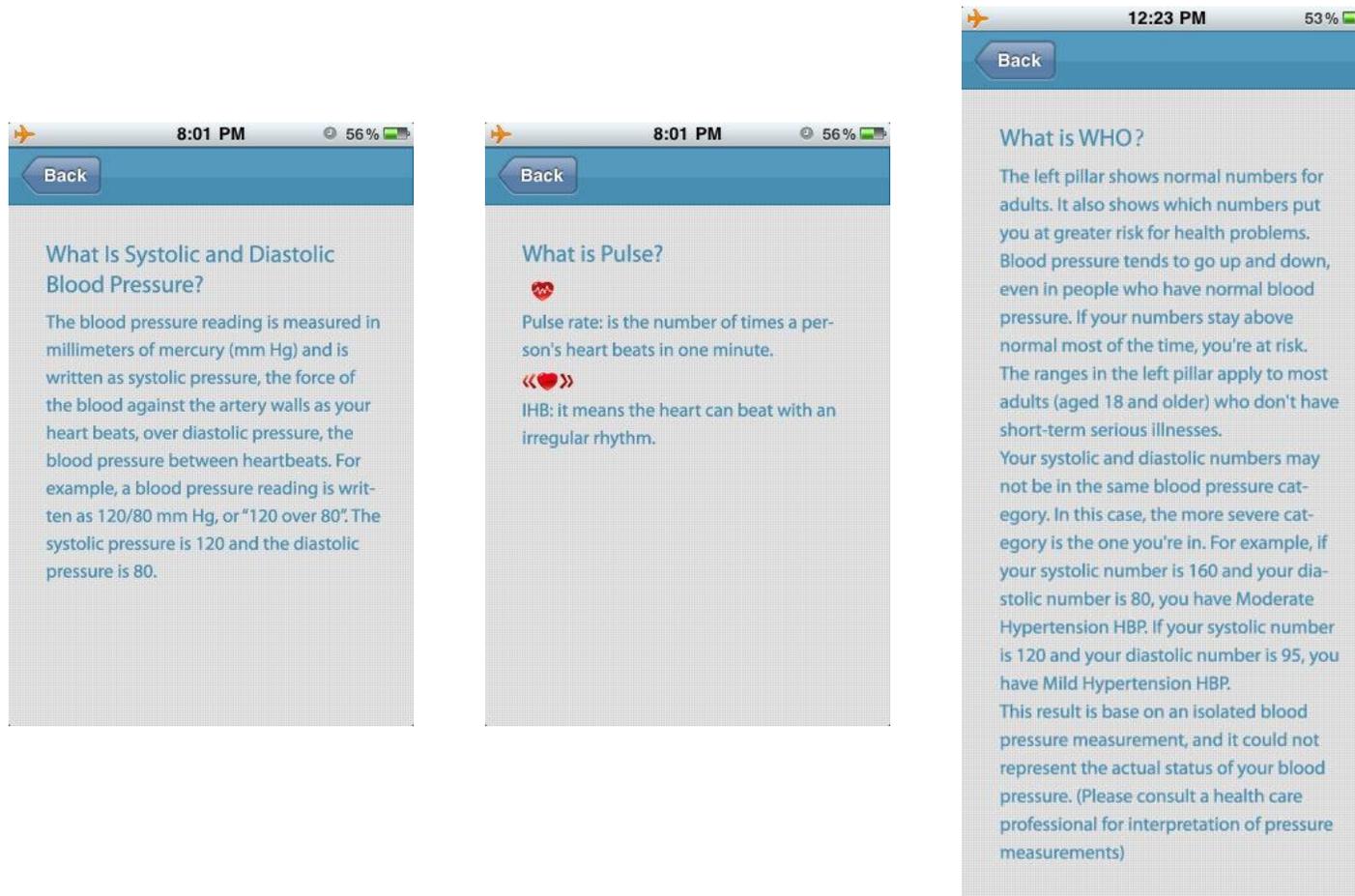

8、 Test Result

You can switch between detail information, WHO classification and pulse wave by slide up-down. And slide left-right for other memories.

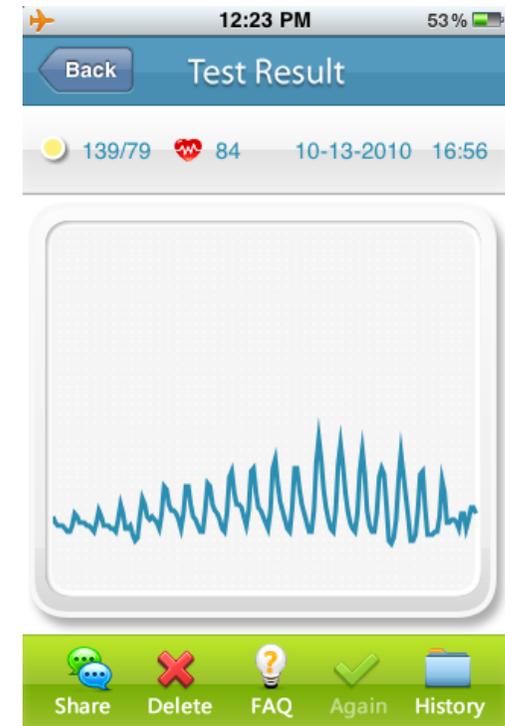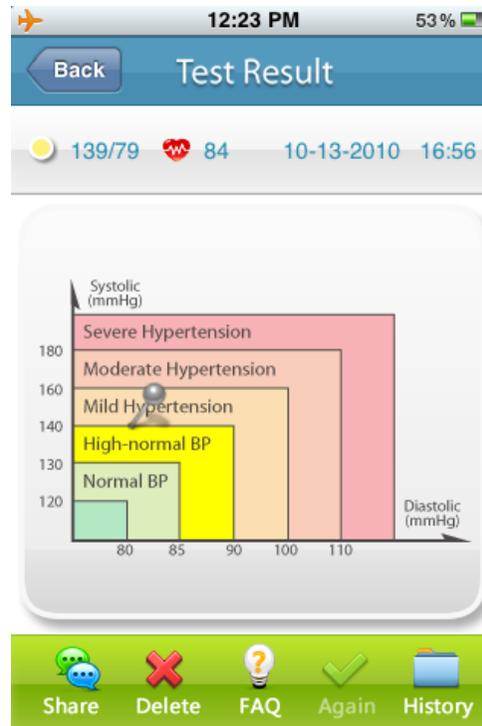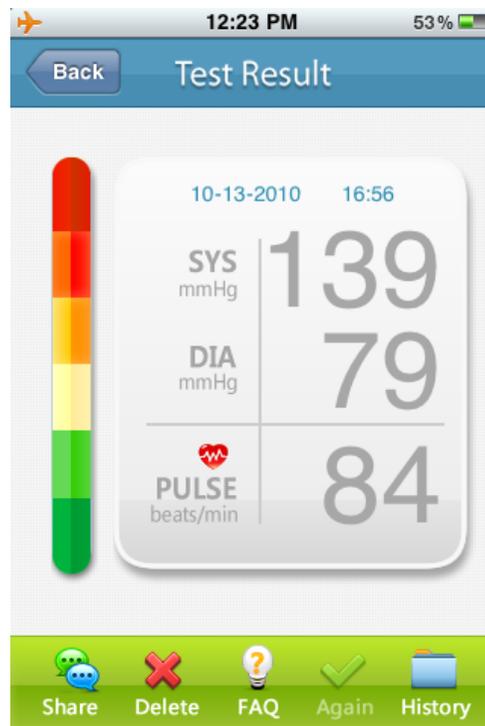

9、 History

Use for review the memory and manage.



Edit: choose for delete memory

Display the WHO classification, Systolic, Diastolic, pulse rate and measurement time. And display the test result was measured or added, the iHealth means measured and pen means added . Touch for go to Test Result page .

- Classify: go to Classify Table page.
- Select: go to Select page for average and index.
- Graph: go to BP Graph page.
- Share: mail the data to the mail address you set.
- Home: go to Main Menu.

10、 BP Graph

Switch this by touch the Graph button in BP Graph page.



Display the detail when touch the column.

The up side of the column is Systolic, and the bottom side is Diastolic. The color is only indicate the classification.


10、 BP Graph

Also can switch this by make the iOS device horizontal in BP Graph page.



11、 WHO Table

Index by WHO classification


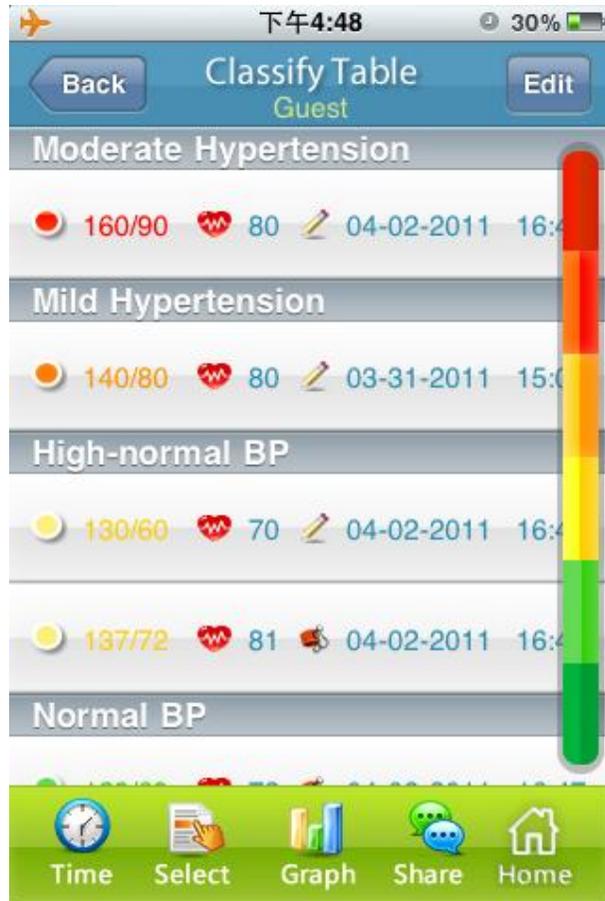


← Classification bar, the following memories is all belongs to this class.

Note: It is not intended to provide a basis of any type of rush toward emergency conditions/diagnosis based on the color scheme and that the color scheme is meant only to discriminate between the different levels of blood pressure.


12、 Classify Graph

Press the Graph button or make the iOS device horizontal in Classify Table page.

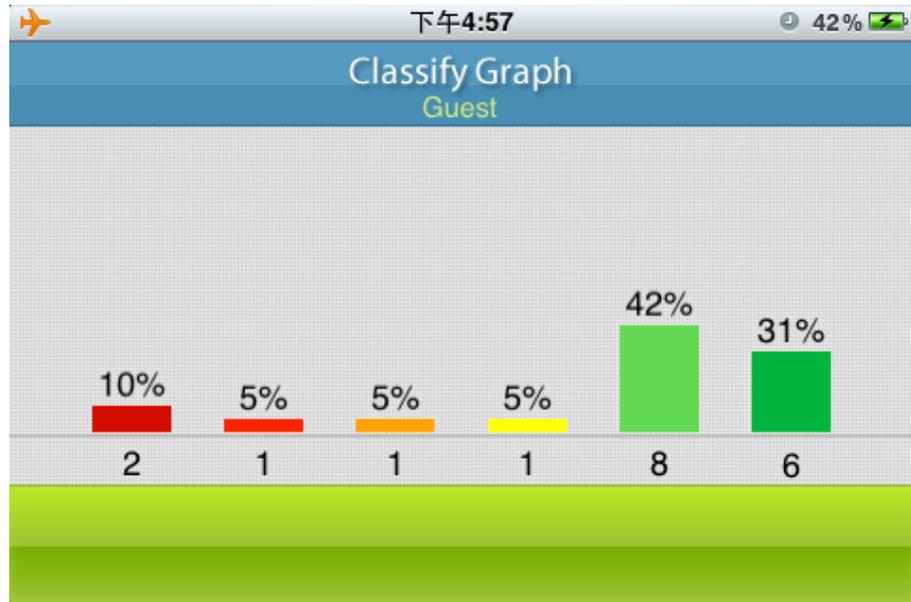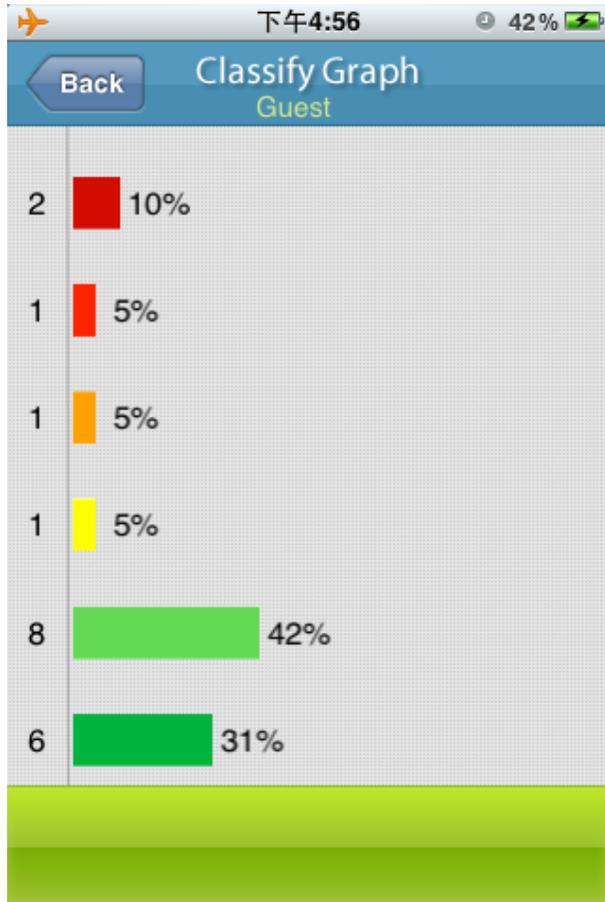

The page shows the classification's corresponding percent of all the memory

13、 Select

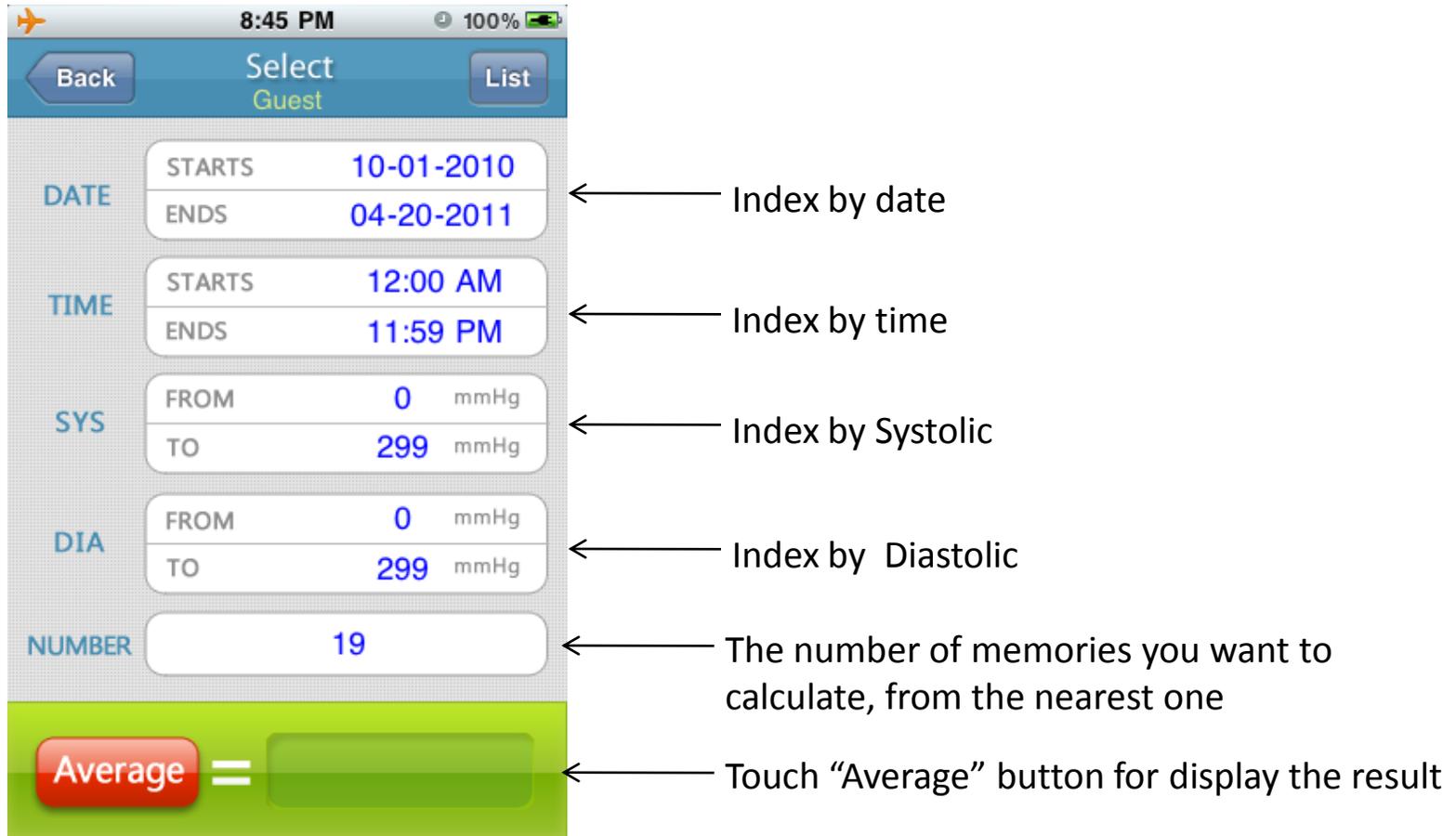


The screenshot shows an iPhone app interface for selecting a date and time range for blood pressure calculation. The interface includes the following fields and annotations:

- DATE**: STARTS 10-01-2010, ENDS 04-20-2011. ← Index by date
- TIME**: STARTS 12:00 AM, ENDS 11:59 PM. ← Index by time
- SYS**: FROM 0 mmHg, TO 299 mmHg. ← Index by Systolic
- DIA**: FROM 0 mmHg, TO 299 mmHg. ← Index by Diastolic
- NUMBER**: 19. ← The number of memories you want to calculate, from the nearest one
- Average** button and result display field. ← Touch "Average" button for display the result

Note: This function is only use for calculate base on chooses option, please consult a health care professional for how to use it.


14、 Information

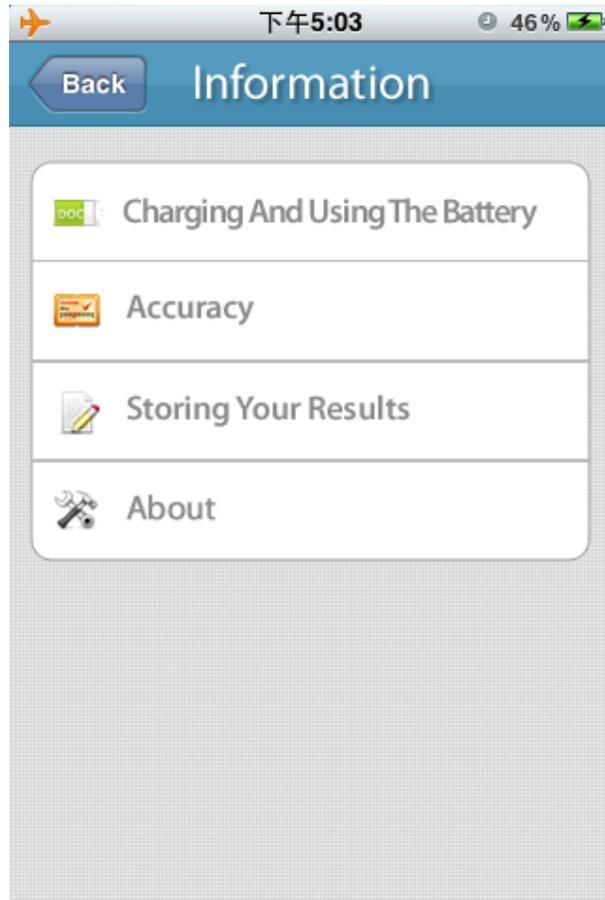

← Touch for go to corresponding page.


<The following content will be occurred in software >

15、 Getting Started

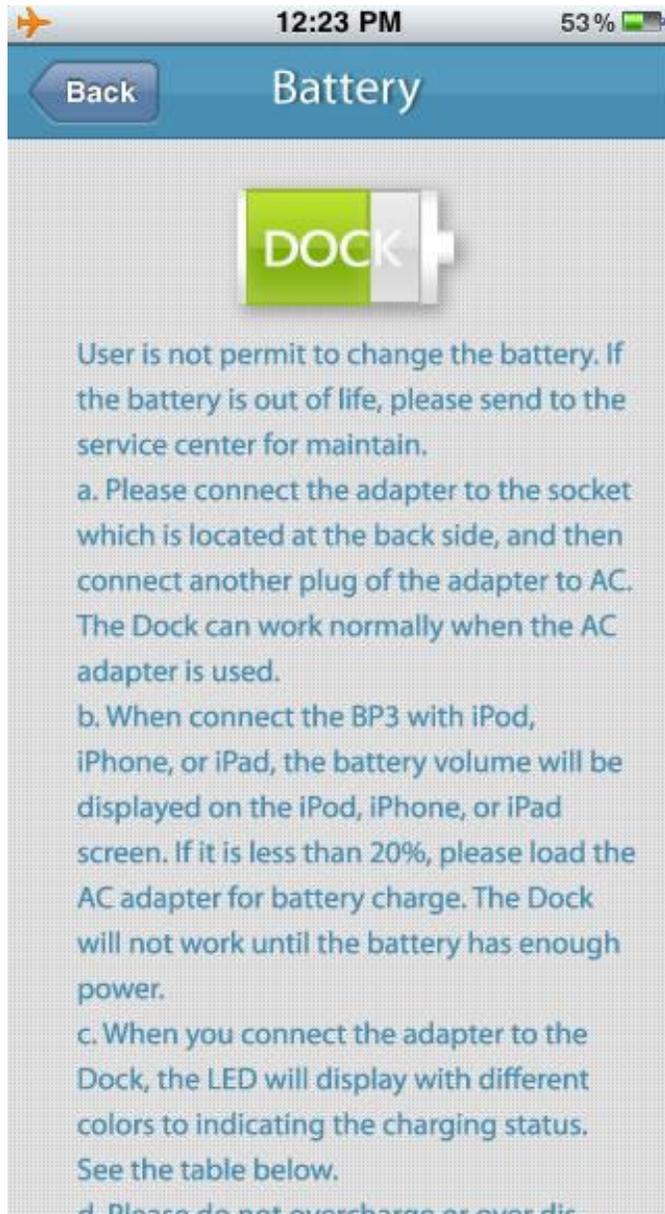


This symbol indicates the current battery capacity for the blood pressure dock. If the dock is unconnected this symbol should be gray.

16、 Accuracy

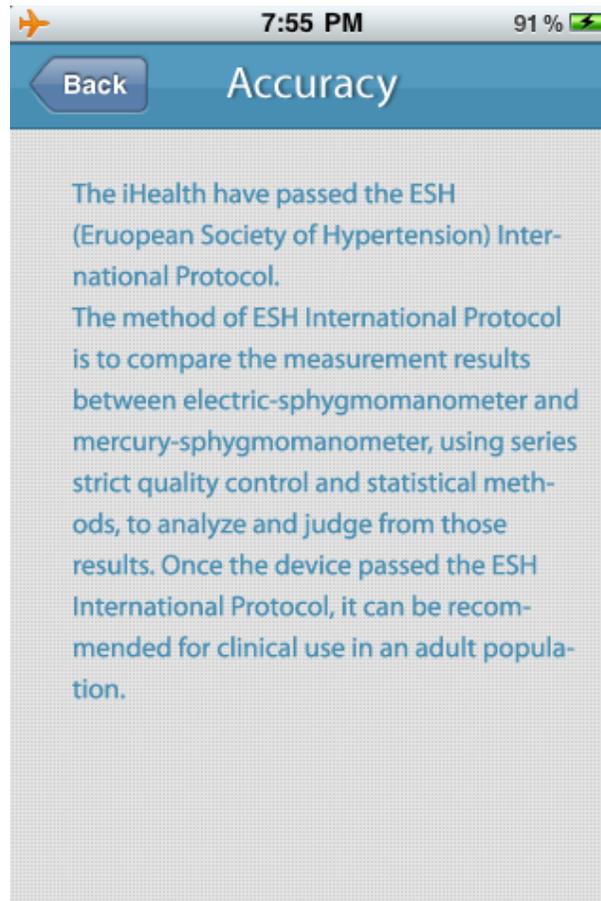


17、 Capacity

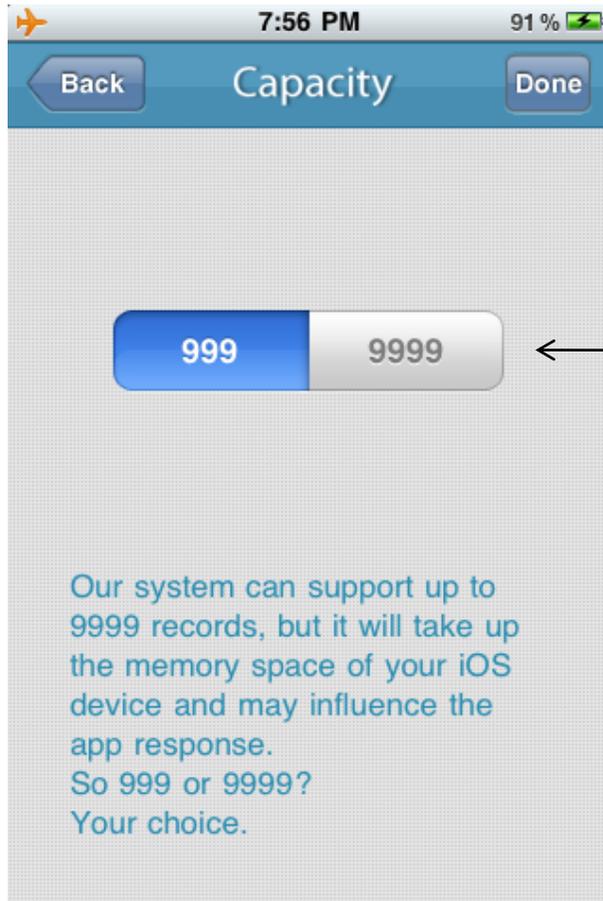


999 and 9999 means the maximum records the system can store. More records means more memory space take up.

18、 About

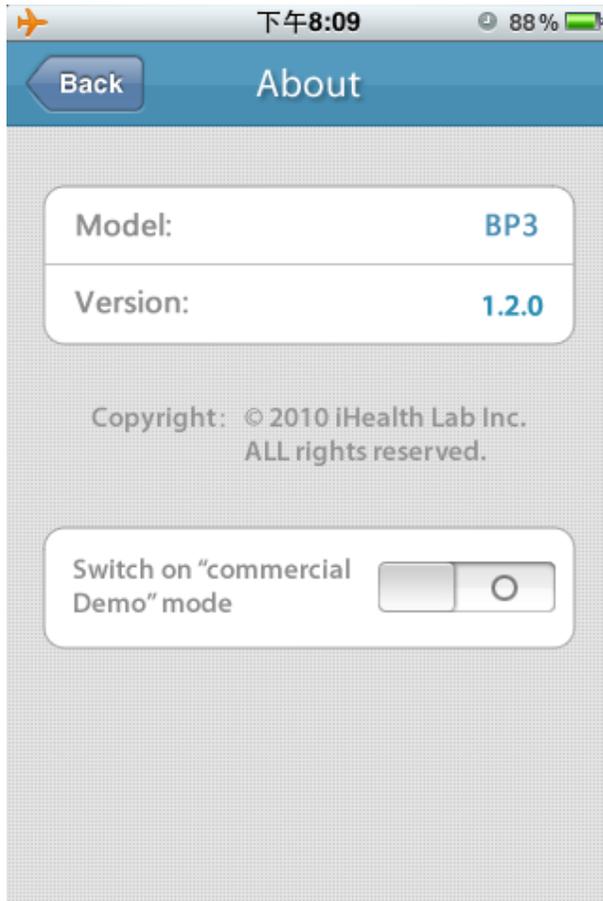

Switch on "commercial Demo" mode

19、 FAQ

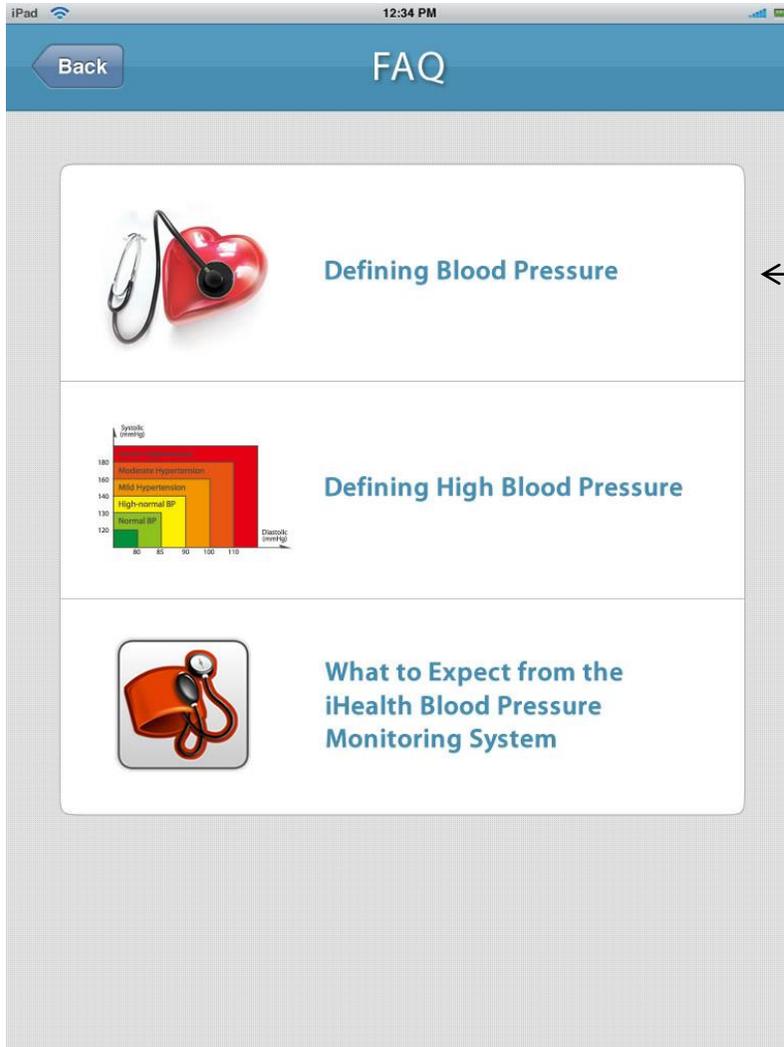

← Touch for go to corresponding page.

